

Med-Flush Airgap – How to Use:


Package Contains:

1 - Chrome Trim Ring

1 - Airgap Receptacle


1 - Airgap

2 - Quick Release Connectors

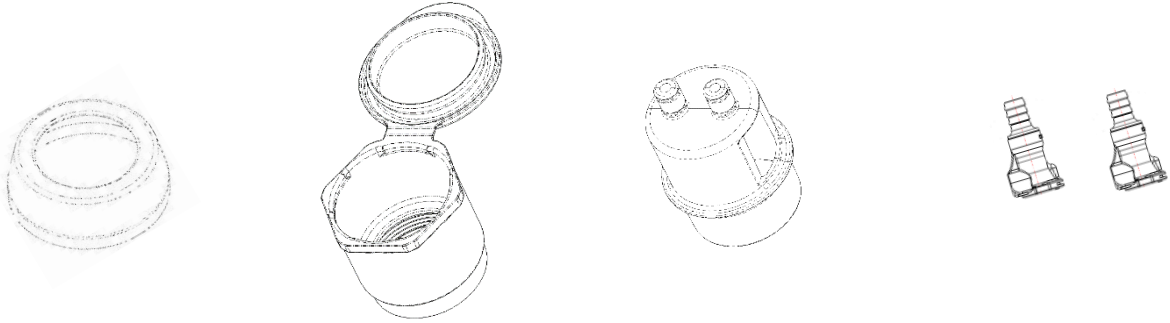


STEP 1:

Place the Chrome Trim Ring over the Threaded Inlet on the Toilet. Then attach threaded end of Airgap Receptacle to the Threaded Inlet. Tighten firmly by Hand to create a good seal. **DO NOT USE TOOLS** to tighten! Make sure flip cap on Airgap Receptacle is open.

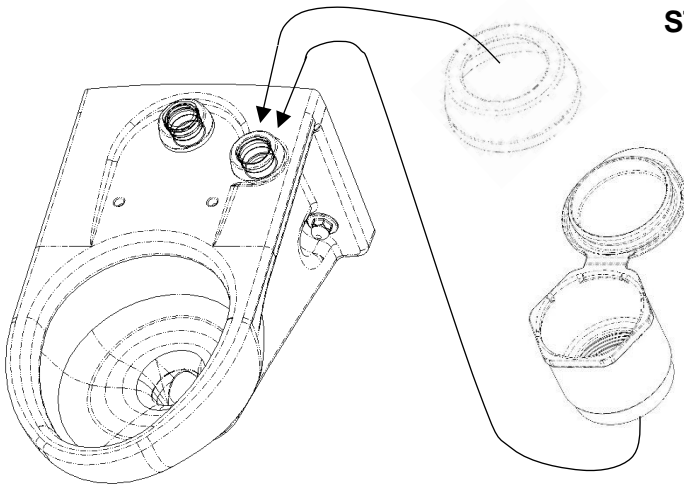


STEP 3:

Push Airgap and Quick Release/Hoses assembly into the Airgap Receptacle until it snaps into place. You may now start the Dialysis Procedure.

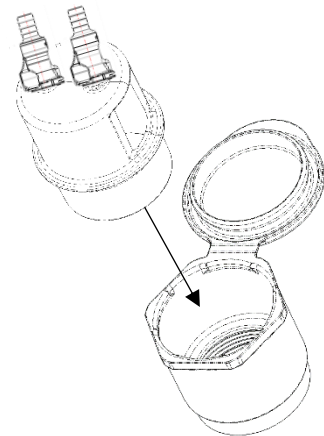

STEP 2:

Dialysis Hoses

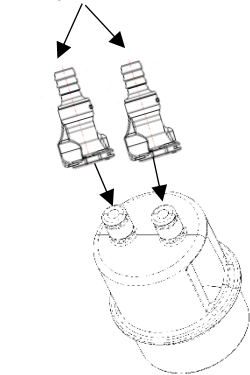

Push Dialysis Hoses onto Male barbed end of Quick Release Connectors, then push Female end of Quick Release Connectors onto Male Quick Release Stems on the Airgap until they snap firmly into place and lock.

STEP 4:

After completing the Dialysis Procedure, pull the Airgap and Quick Release/Hoses assembly from the Airgap Receptacle. Close the flip cap on the Airgap Receptacle. Remove both Quick Release Connectors from the Airgap and dispose of the Airgap.